pH Test Strips

CZ

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ENG

Do you want to know your body's pH, but you don't know how to find it? It is really simple!

Remedicum pH test strips will help you find out whether your body is acid, or is at the optimal pH. And that can be done in the comfort of your home.

- to test urine or saliva
- pH range 4.5 – 9.0
- measurements by 0.25 parts (degrees)
- cost-effective package of 80 pcs
- simple and quick measurement

Instructions for use:

To test from urine: for testing, use the first and second flow of urine of the day. Take the average of the measured results. Insert the test strip for a short time into your urine stream or dip it into a cup containing your urine sample, slightly knock off the excessive urine and put the paper on a horizontal surface. Wait for 15 seconds and compare the test strip with the colour pH scale to find out your pH.

To test from saliva: perform the test 2 hours after your meal. Swallow the saliva in your mouth and create new saliva. Spit it out onto a plastic spoon (do not put the test strip directly into your mouth) and dip the test strip into the saliva. Wait for 15 seconds and compare the test strip with the colour pH scale to find out your pH.

Measurement results:

pH	6.75 – 7.25	optimal
pH	6.5	slightly acid
pH	6.0 – 6.25	acid
pH	below 6.0	very acid

Note:

1. Urine is slightly more acid than saliva. Subtract 0.25 from the results of your measurement from saliva.
2. The colour test scale has units of 0.25, so if your pH is, for example, 6.85, your test strip will have the colour of the part between pH 6.75 and pH 7.0. In such a case, choose the higher result (pH 7.0) to determine your pH.
3. Some substances in your urine or saliva, such as alcohol or drugs, can slightly alter the result of your measurement. The bottom colour square is rather more sensitive to this. Therefore, if the top colour square is identical with the top colour square at the colour test scale, and the bottom square does not correspond to the bottom square of the test scale, then use the result measured at the top square to determine your pH.

Hyperacidity of organism:

- you have little energy
- you feel tired
- you cannot reduce your weight
- you suffer from various illnesses and pains

Hyperacidity of organism leads to demineralization of bones, teeth, joints, and has negative impact on your urinary and digestive system. An acid environment in your body produces fertile ground for microorganisms, which can impair your health in the future.

Hyperacidity of your organism can also make you feel tired, gain weight, have problems with your digestion, and experience aches and pains of various kinds.

Note:

human body's pH should be slightly alkaline – around 7.4. pH of blood 7.365 – 7.45.

pH zone	pH zone	pH zone	pH zone
4,5 – 5,75	6,0 – 6,5	6,75 – 7,25	7,5 – 9,0
very acid	acid	optimal	too alkaline
hyperacidity of organism (50-100 times more acid)	the body is 5-10 times more acid.	CONGRATULATIONS!	Your body seems to be producing ammonia, which is alkaline, in order to get rid of hyperacidity.
Complete body cleansing, detoxification of the organism, alkalisation (mineralization of the body). Complete change of eating habits.	Modification of eating habits (adding green groceries) alkalizing food supplements.		

Important:

Dispose of the used test strip in accordance with hygienic standards. Do not throw it into the toilet. Do not touch the colour test squares on the test strip. For external use only (in vitro). Intended for self-testing. Keep out of the reach of children. Store at room temperature in a cool and dry place. For single use only. Do not use the test strips after the expiry date, which is printed on the package.

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